NTI-Kahla GmbH • Rotary Dental Instruments
 Im Camisch 3 • D-07768 Kahla/Germany
 Tel. +49(0)36 424-573-0 • Fax +49(0)36 424-573-29
 e-mail: nti@nti.de • http://www.nti.de

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GB Root Post Product Information

This user information relates to the following
 titanium alloy post and core systems:

passive and active HeadMaster system
passive PostMaster system
active PinMaster system.

Read through the product information carefully
 before using the post and core system. The manu-
 facturer does not accept any liability for damage
 caused by not adhering to these guidelines or
 improper use of the system.

Contents:

1. Areas of application for the post and core
 systems
2. Disinfecting and cleaning
3. Recommended uses

1. Area of application for the post and core systems

The post and core systems were developed for
 building up teeth with missing crown sections.
 The post and core systems are used after root
 treatment has been successfully completed.

2. Disinfecting, cleaning and sterilising

Rotary instruments should be disinfected in an
 approved sterilising solution according to the
 manufacturer's instructions before using for the
 first time and immediately after each use. They
 should then be rinsed and dried. Sterilisation is
 carried out in an autoclave at 134°C / 2.1bar
 for 5 min. (or according to the autoclave
 manufacturer's instructions).

3. Recommended uses

The post and core systems are inserted using
 low viscosity phosphate cement.
 The following stages are the same for all post
 and core systems.

1. Completed endodontic treatment
2. Removal of the root filling
3. Prepare the root with a reamer.
4. Smooth the root face, e.g. with a round
 diamond rotary instrument.
5. Select the correct size of post using the X-ray
 image.
6. Drill to a depth of approx. 1 mm with a root-
 facing drill using light pressure (rpm: 2,000
 min⁻¹) – **only necessary for posts with a head
 (HeadMaster).**
7. Determine the required drilling depth using the
 X-ray image and mark with an endo stop.
8. Drill the hole to the preset depth (rpm: 500-
 1,000 min⁻¹).
9. Clean and disinfect the root canal.
10. Place the post onto the insertion key.

Procedure with passive HeadMaster posts with a head and passive PostMaster posts without a head:

11. Fill the canal with cement.
12. Apply cement to the shank of the post.
13. Insert the post into the canal using light
 pressure.
14. After the cement has hardened, additional
 retention can be provided by a parapulpal pin.
15. Fabricate a core using a resin filling material.

Procedure with active HeadMaster posts with a head and active PinMaster posts without a head:

11. Thread the post carefully into the canal.
12. After two turns rewind the post back once.
 This prevents the post jamming and reduces
 the risk of the root fracturing. Thread the post
 in until it reaches the final position and then
 unscrew it.
13. Apply cement to the shank of the post.
14. Fill the canal with cement.
15. Thread the active post carefully into the canal.
16. After the cement has hardened, additional
 retention can be provided by a parapulpal pin.
17. Fabricate a core using a resin filling material.

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© 2006 NTI-Kahla GmbH Rotary Dental Instruments